



Mintbowl inc.
Mob: +91.997.903.5679
www.dogwheelchairsindia.com
info@dogwheelchairsindia.com

Made with love



Assembly Height Adjustment



find these four allen-key bolts backside of the wheelchair to adjust the wheelchair height. Just use your allen-key tool to adjust the height of your wheelchair according to your requirement.

Width Adjustment

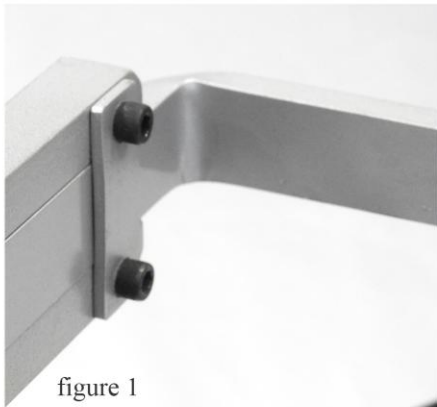


figure 1

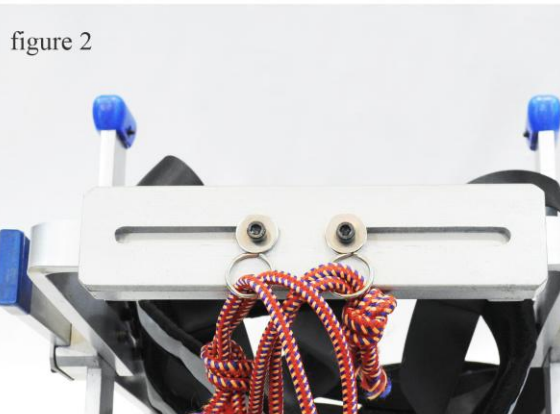


figure 2

To adjust your wheelchair width you need to adjust these allen-key bolt shown in figure 1 and figure 2 which you can find back-side of the wheelchair.

Assembly Length Adjustment



To adjust the length of your wheelchair you need to adjust these two allen-key bolts according to your dog length shown in **figure 1** if u have Large size wheelchair. if you have medium size wheelchair then you need to adjust **allen-grub-screw** (hole upside the mintbowl logo on wheelchair) shown in figure 2 in red square.



Tight disable back legs of your dog same like given picture in between the velcro belt. open the velcro belt and put your dog leg in the belt and tight it again and finally tight the rope with steel cap.